


Endoscopic Twin Forceps


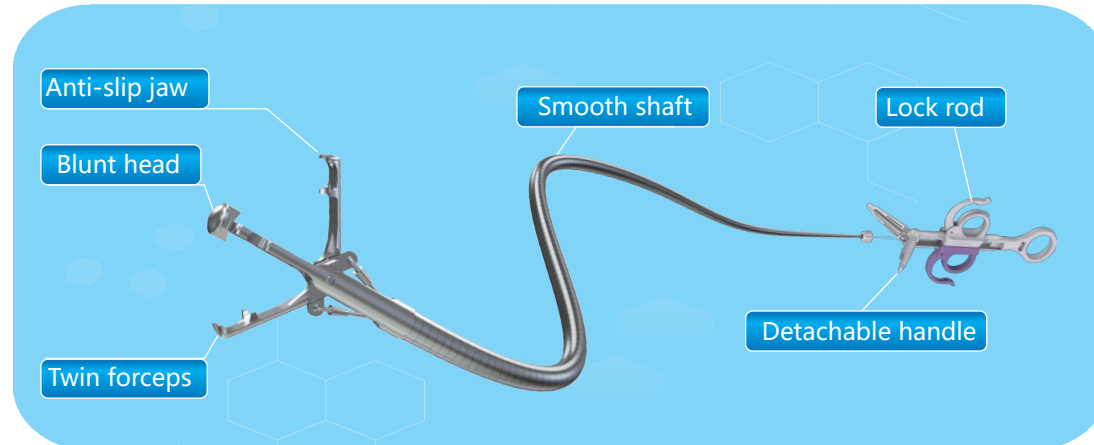


Specifications	Head diameter(mm)	Shaft length(mm)	Jaw max opening angle
SS211-2312	2.3	1200	≥90°
SS211-2314	2.3	1400	≥90°
SS211-2316	2.3	1600	≥90°
SS211-2318	2.3	1800	≥90°
SS211-2323	2.3	2300	≥90°

Protector Cap



Specifications	Max outer diameter(mm)
SS270-15-A	15
SS270-17-A	17

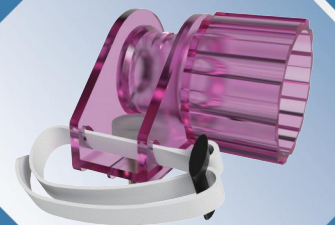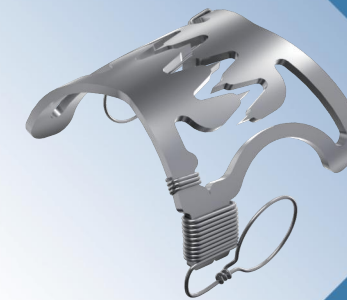







Detachable

Endoscopic
Anastomosis Clip


Indications

The Endoscopic Anastomosis Clip is indicated for use in flexible endoscopy and for the compression of tissues in the gastrointestinal tract, for hemostasis or for treating lesions of the wall of gastrointestinal tract.

Usages

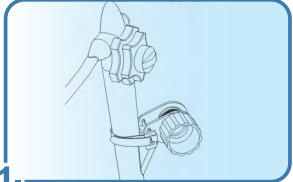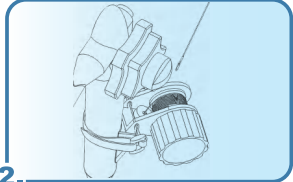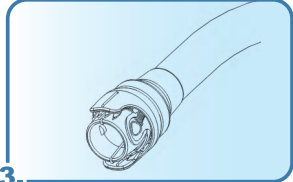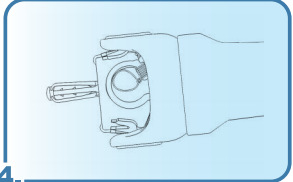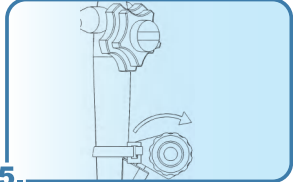
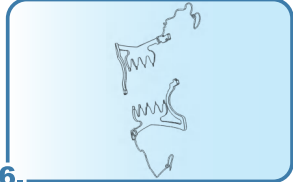
-  Remove the biopsy cap and install the Releaser
-  Insert wire hook to pull threads and then wrap it around the reel
-  Install applicator cap on endoscope tip
-  Grasp tissues into cap by suction or twin forceps assistance
-  Rotate reel rapidly to release the clip
-  To remove clip by untying binding wire

Clinical application

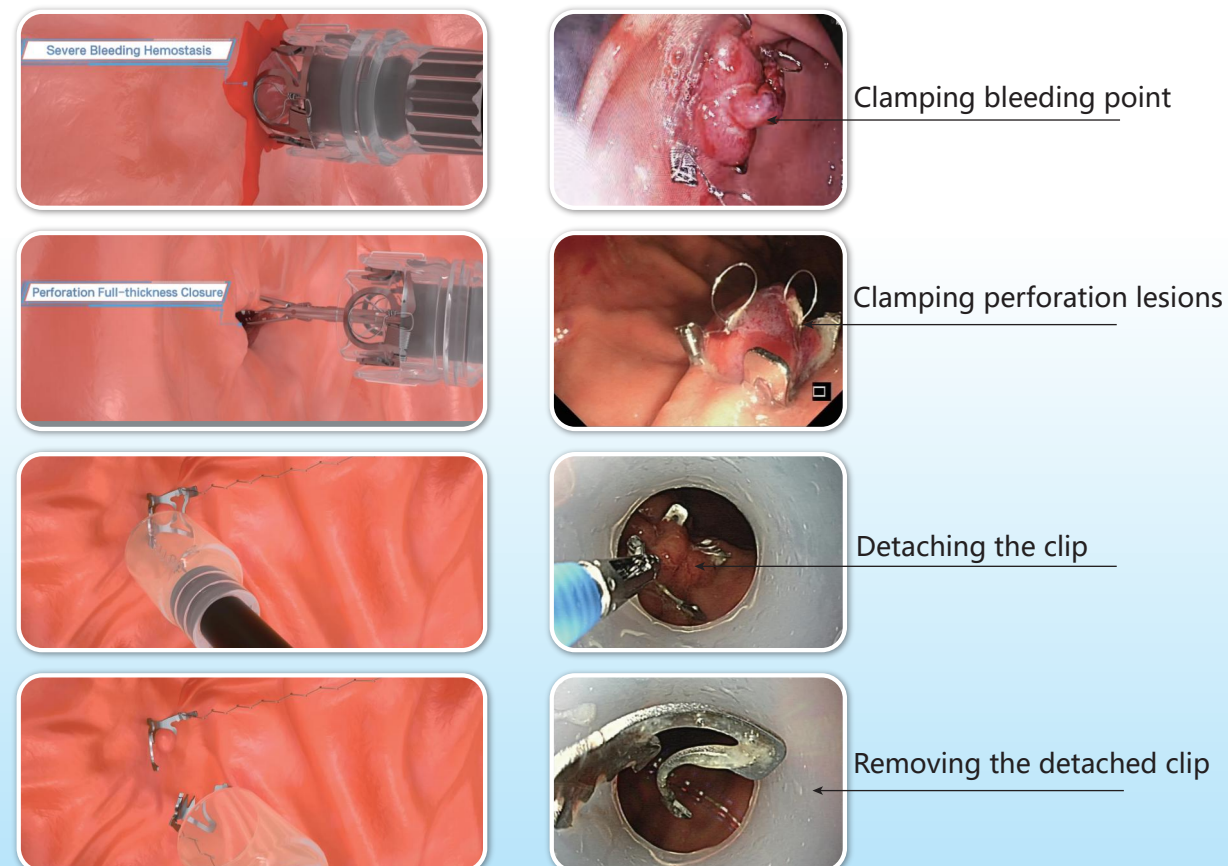


Product features

Detachable endoscopic clip, Strong clamping force, Wide clamping range

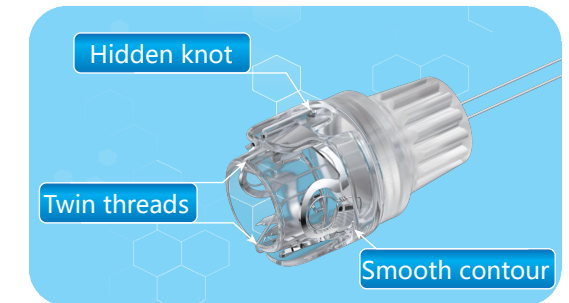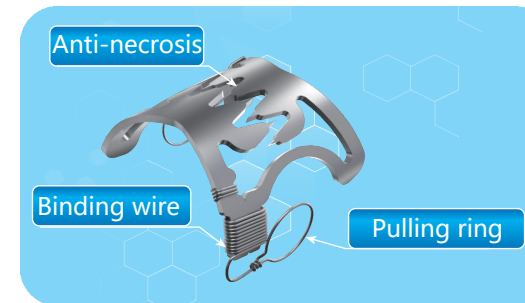

Specifications

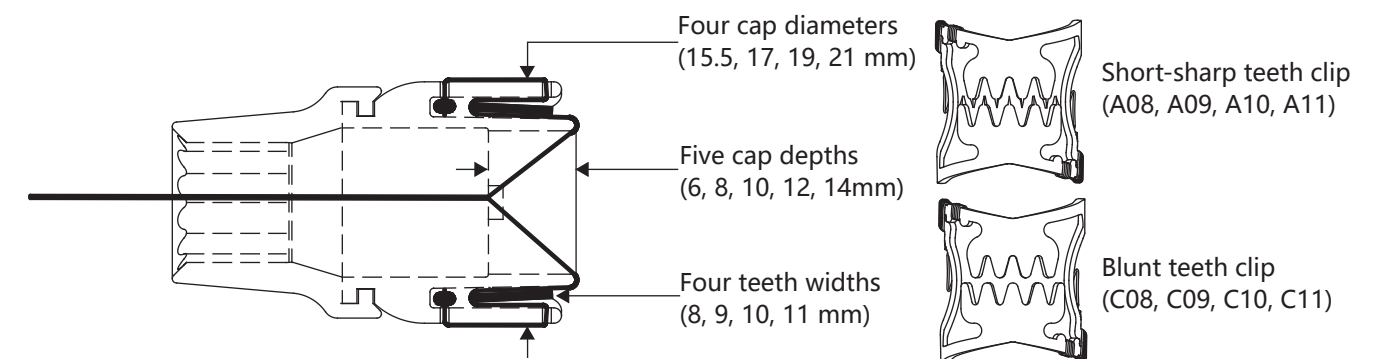

Specifications	Cap diameters (mm)	Cap depths (mm)	Clip type	Matched endoscope tip diameter (mm)	Closure scope (mm)
SI241-1506-A08	15.5	6	A08	8~10.2	0~15
SI241-1508-A08	15.5	8			
SI241-1706-A09	17	6	A09	8.5~11	0~20
SI241-1710-A09	17	10			
SI241-1906-A10	19	6	A10	10~13	0~25
SI241-1912-A10	19	12			
SI241-2106-A11	21	6	A11	12~14	0~35
SI241-2114-A11	21	14			

Notes:

The SI241 series products contain one releaser, one applicator cap, one anastomosis clip, two wire hook and one protector cap. The SI244 series products are supplementary package for extra applicator cap with anastomosis clip. The anastomosis clip is divided into four types (A, B, C, D) based on different teeth shapes. Type A clip is short-sharp teeth clip which suits for general tissue, Type B clip is long-sharp teeth clip which suit for fibrotic tissue. Type C clip is blunt teeth clip which suits for fragile tissue, Type D clip is lateral teeth clip which suits for smooth tissue. Both the type A, type B, type C clip are the same teeth width, Type D clip has an extended lateral teeth particularly.